



# polishing paste

ENGLISH

## INSTRUCTIONS FOR USE

The SDI Polishing Paste imparts the appearance of a microfilled to the high strength, anterior / posterior hybrid composite. It contains agglomerated primary crystals of aluminium oxide particles that impart a smooth, luminous finish to all resin surfaces.

### COMPOSITION:

Glycerine  
Aluminium oxide

### INDICATIONS:

Suitable for composite, compomer and amalgam polishing. Porcelain, ceramic and gold finishing.

### INSTRUCTIONS:

1. Apply the SDI polishing paste to a rubber cup or disc.
2. From the rubber cup or disc, smear a thin layer of the paste onto the restoration.
3. Add a small amount of water to both the tooth and cup or disc.
4. Polish for 30 seconds at low speed and light pressure. SDI Polishing Paste may be used interproximally with floss.
5. Wash and dry.

**WARNING:** Apply polishing paste at normal room temperature (23°C/74°F).

### PRECAUTIONS:

Use only as a final polish.  
Keep out of reach of children.  
Do not take internally.  
Store at temperatures between 10° and 25°C (50°-77°F).  
When not in use replace the cap tightly.  
Do not use after expiry date.  
Caution: Federal Law restricts this device to sale by on the order of a dentist.  
Date of issue of last revision : February 2010

### Polishing Paste

Reorder 8100150

4g syringe



Made in Australia by SDI Limited  
Bayswater, Victoria 3153  
Australia 1 800 337 003  
Austria 00800 022 55 734  
Brazil 0800 770 1735  
France 00800 022 55 734  
Germany 0800 100 5759  
Ireland 01 886 9577  
Italy 800 780625  
New Zealand 0800 734 034  
Spain 00800 022 55 734  
United Kingdom 00800 022 55 734  
USA & Canada 1 800 228 5166

EC REP

SDI Dental Limited  
Block 8, St Johns Court  
Swords Road, Santry  
Dublin 9, Ireland  
Tel +353 1 886 9577  
www.sdi.com.au



PAK95053 E