

SDI

riva coat

**E
N
G
L
I
S
H**

LIGHT CURED COATING MATERIAL FOR GLASS IONOMER PRODUCTS INSTRUCTIONS FOR USE

Riva Coat is designed for the prevention of saliva and water contamination to exposed surfaces of glass ionomer cements during the first 24 hours immediately after placement.

INDICATIONS:

Glass ionomer cement restorations requiring a coating material after placement.

CONTRAINDICATIONS:

Product may cause skin irritations to some people. In such cases, discontinue use and seek medical attention.

INSTRUCTIONS FOR USE:

1. Place the glass ionomer cement according to manufacturer's instructions.
2. When restoration has hardened, apply Riva Coat to all exposed surfaces of restoration.
3. Light cure for 20 seconds.
4. Replace cap immediately after use.

STORAGE AND HANDLING

Material is light sensitive. Do not dispense Riva Coat until ready to use. Store in a dark place at temperatures between 10° and 25°C (50° - 77°F). Use at room temperature between 20° and 25°C (68° - 77°F). Shelf life: 3 years

PRECAUTIONS:

For professional use only.

Keep out of reach of children.

Do not take internally.

Do not use after expiry date.

Caution: Federal Law restricts this device to sale by or on the order of a dentist.

MSDS available at www.sdi.com.au or contact your regional representative.

FIRST AID

Eye (contact): Wash thoroughly with water and seek medical attention.

Skin (contact): Remove using a cloth or sponge soaked in alcohol / acetone. Wash thoroughly with water.

Ingestion: Drink plenty of water. Seek medical attention if symptoms persist.

Inhalation: No symptoms expected.

Date of issue of last revision : April 2010

PACKAGING

5mL bottle refill



Made in Australia by SDI Limited

Bayswater, Victoria 3153

Australia 1 800 337 003

Austria 00800 022 55 734

Brazil 0800 770 1735

France 00800 022 55 734

Germany 0800 100 5759

Ireland 01 886 9577

Italy 800 780625

New Zealand 0800 734 034

Spain 00800 022 55 734

United Kingdom 00800 022 55 734

USA & Canada 1 800 228 5166

www.sdi.com.au

EC	REP
----	-----

SDI Dental Limited
Block 8, St Johns Court
Swords Road, Santry
Dublin 9, Ireland
Tel +353 1 886 9577

CE
0297

10°-25°C
50°-77°F