

SDI**riva conditioner****E
N
G
L
I
S
H****POLYACRYLIC ACID
CONDITIONER
INSTRUCTIONS FOR USE**

Riva Conditioner is a mild solution of polyacrylic acid developed to partially remove the smear layer of dentin and enamel. This optimizes adhesion of the glass ionomer cement to tooth structure.

INDICATIONS:

Glass ionomer cement restorations requiring a conditioning material prior to application.

CONTRAINDICATIONS:

Product may cause skin irritations to some people. In such cases, discontinue use and seek medical attention.

INSTRUCTIONS FOR USE:

1. After tooth preparation, apply Riva Conditioner to the prepared surfaces.
2. Leave for 10 seconds.
3. Rinse thoroughly with water.
4. Remove excess water. Keep moist. Avoid contamination.
IMPORTANT: Be careful not to completely dry the surface.
5. Apply glass ionomer according to manufacturer's instructions.
6. Replace cap immediately after use.

STORAGE AND HANDLING

Do not dispense Riva Conditioner until ready to use.

Store in a dark place at temperatures between 10° and 25°C (50° - 77°F).

Use at room temperature between 20° and 25°C (68° - 77°F).

Shelf life: 3 years

PRECAUTIONS:

For professional use only.

Keep out of reach of children.

Do not take internally.

Do not use after expiry date.

Caution: Federal Law restricts this device to sale by or on the order of a dentist.

MSDS available at www.sdi.com.au or contact your regional representative.

Date of issue of last revision : May 2010

FIRST AID

Eye (contact): Wash thoroughly with water and seek medical attention.

Skin (contact): Wash thoroughly with water.

Ingestion: Drink plenty of water/milk. Seek medical attention if symptoms persist.

Inhalation: No symptoms expected.

PACKAGING

10mL bottle refill

Made in Australia by SDI Limited
Bayswater, Victoria 3153

Australia 1 800 337 003

Austria 00800 022 55 734

Brazil 0800 770 1735

France 00800 022 55 734

Germany 0800 100 5759

Ireland 01 886 9577

Italy 800 780625

New Zealand 0800 734 034

Spain 00800 022 55 734

United Kingdom 00800 022 55 734

USA & Canada 1 800 228 5166

www.sdi.com.au

EC REP

SDI Dental Limited
Block 8, St Johns Court
Swords Road, Santry
Dublin 9, Ireland
Tel +353 1 886 9577

CE
0297

10°-25°C
50°-77°F