



soothe

E
N
G
L
I
S
H

INSTRUCTIONS FOR USE

Soothe is a topical desensitising gel containing 6% potassium nitrate and 0.10% fluoride ions. It is designed to relieve tooth sensitivity caused by chemical and thermal changes.

COMPOSITION:

Potassium Nitrate	6.0%
Fluoride ions	0.10%
Water	89.60%
Thickener	4.20%
Sodium benzoate	0.10%

INSTRUCTIONS:

- Rub onto the affected area for 2 minutes.
- Do not eat or drink for at least 10 minutes.
- Repeat every 2 - 3 hours until symptoms have subsided.

PRECAUTION:

To be used under the supervision of a dental professional.
Do not use if you have any known allergies to the listed composition.

Keep product out of heat and sunlight.

Store at room temperature.

Excessive amounts should not be swallowed by children with developing teeth.

Keep out of reach of children

CAUTION:

Federal law restricts this device to sale by or on the order of a dentist.

Date of issue of last revision : March 2010



Made in Australia by SDI Limited
Bayswater, Victoria 3153
Australia 1 800 337 003
Austria 00800 022 55 734
Brazil 0800 770 1735
France 00800 022 55 734
Germany 0800 100 5759
Ireland 01 886 9577
Italy 800 780625
New Zealand 0800 734 034
Spain 00800 022 55 734
United Kingdom 00800 022 55 734
USA & Canada 1 800 228 5166
www.sdi.com.au

EC REP

SDI Dental Limited
Block 8, St Johns Court
Swords Road, Santry
Dublin 9, Ireland
Tel +353 1 886 9577



PAK41067 E