For better dentistry

DENTSPLY

Compules[®] Tips Gun

Applicator Gun

Applikationspistole

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Dispenser a pistola

Pistola de aplicaci3n

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CAUTION: For dental use only.

USA: Rx only.

| Content | Page |
|-----------------------------------|------|
| 1 Product description..... | 2 |
| 2 Safety notes | 2 |
| 3 Step-by-step instructions | 3 |
| 4 Hygiene..... | 3 |
| 5 Lot number and durability..... | 4 |

1 Product description

Compules[®] Tips Gun is an applicator gun for all DENTSPLY filling materials delivered in Compules[®] Tips and Compula[®] Tips.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- The Compules[®] Tips Gun is only intended to be used with DENTSPLY Compules[®] or Compula[®] Tips.
- The material should extrude easily. **DO NOT USE EXCESSIVE FORCE.** Excessive pressure may cause the Compules[®] Tip to rupture or to eject from the Compules[®] Tips Gun.
- Please also refer to the Instructions for Use of the products used with this product.

3 Step-by-step instructions

3.1 Application

1. Insert Compules® Tip into the notched opening of the Compules® Tips Gun. Be certain that the collar on the Compules® Tip is inserted first.
2. Remove the colored cap from the Compules® Tip. The Compules® Tip may be rotated 360° to gain the proper angle of entrance into the cavity or to the mixing pad. **DO NOT USE EXCESSIVE FORCE.**
Dispense the material directly into the cavity preparation using a slow, steady pressure.
3. To remove the used Compules® Tip, be sure that the Compules® Tips Gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compules® Tip and remove.

Excessive force.

Injury.

1. Apply slow and steady pressure on the Compules® Tips Gun.
2. Do not use excessive force. Compules® Tip rupture or ejection from Compules® Tips Gun may result.

For further information please refer to composite Instruction for Use.

4 Hygiene

NOTICE

Wrong cleaning or disinfection method.

Damage to the device.

1. Always disassemble Compules® Tips Gun before reprocessing.
2. Do not clean or disinfect with aggressive agents (e.g. solutions based on orange oil or acetone).

4.1 Disassembly and preparation for reprocessing

1. Partially close the Compules® Tips Gun and place your thumb on the rear part of the hinge.
2. Push upwards and lift hinge. The Compules® Tips Gun is separated into two parts and the plunger is exposed.
3. In case of gross contamination of the Compules® Tips Gun, remove it using a soft brush and running water.
4. Remove restorative material, if any, with a soft paper tissue and alcohol of 70% vol.

4.2 Manual cleaning and disinfection

1. Discard gloves according to local regulations.
2. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
3. Use a new clean pair of examination gloves.
4. Clean Compules® Tips Gun with a soft brush, warm water, and an appropriate cleaning solution until free of visible contamination. Pay special attention to device seams and insertions.
5. Remove cleaning solution residue with a cloth soaked with water.

6. Dry device with a lint-free single-use cloth.
7. Thoroughly wipe all surfaces of the Compules® Tips Gun with a single-use cloth in combination with an alcohol-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.
8. Remove disinfectant solution residue with a cloth soaked with water.
9. Dry Compules® Tips Gun with a lint-free single-use cloth.

4.3 Automatic Washer-Disinfector

As an alternative to manual cleaning and disinfection, the Compules® Tips Gun can also be processed in a washer-disinfector¹. Use only properly maintained, calibrated, and approved washer-disinfector according to ISO 15883-1.

1. Place Compules® Tips Gun in the washer-disinfector allowing water and detergent to enter into and drain off from Compules® Tips Gun openings during washer-disinfection program and follow washer-disinfector's Instructions for Use.
2. Run washer-disinfection program with A0 value ≥ 3000 (e.g. 5 min at ≥ 90 °C) using appropriate detergents¹.

4.4 Autoclaving (optional)

The Compules® Tips Gun can also be autoclaved.

1. Steam autoclave device unwrapped at 134 °C/ 2 bar = 273 °F/29 psi for 3 minutes 30 seconds.

4.5 Reassembly

1. After reprocessing insert plunger into the Compules® Tips Gun cylinder.
2. Press components together and snap hinge mechanism in place.

Do not continue to use damaged or soiled Compules® Tips Guns.

Compules® Tips Gun can be reprocessed for up to 200 times.

5 LOT number (LOT) and durability

1. Do not use Compules® Tips Gun which is reprocessed for more than 200 times or in use for longer than two years.
2. The following numbers should be quoted in all correspondence:
 - Reorder Number
 - Lot number

© DENTSPLY DeTREY 2015-08-31

[These Instructions for Use are based on Master Version 02]

¹ For example Miele Washer-Disinfector G7835 CD using program Vario TD with neodisher® Medi-Clean (alkaline detergent) and neodisher® Z (acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany (no products or registered trademarks of DENTSPLY International).

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DENTSPLY DeTREY GmbH
DENTSPLY Limited
DENTSPLY Italia S.r.l.
DENTSPLY DeTrey GmbH