Instructions for Use

**DENTSPLY**

**MULTI**   
**Enhance**<sup>®</sup>

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## Multiple use Finishing & Polishing System

**CAUTION:** For dental use only.

**In the USA:** Rx only

Content	Page
1 <b>Product description</b> .....	2
2 <b>General safety notes</b> .....	3
3 <b>Step-by-step instructions</b> .....	4
4 <b>Hygiene</b> .....	4
5 <b>Lot number and expiration date</b> .....	5

### 1 Product description

The Enhance® Multi Finishers and Polishers are multiple use, diamond impregnated, silicone Discs, Cups and Points designed for preparing and polishing the surface of composite and compomer restorations in a two step technique.

#### 1.1 Delivery forms

- Assortment of Enhance Multi Finishers, Polishers and accessories plus organizer tray.
- Refill packages containing 5 instruments of the same type each.

#### 1.2 Composition

Active part: Diamond grit, synthetic rubber, pigments  
 Shaft: Stainless steel

#### 1.3 Indications for use

- The dark grey Enhance Multi Finishers are indicated for contouring and pre-polishing of composite and compomer restorations.
- The light grey Enhance Multi Polishers are indicated for polishing of composite and compomer restorations.

#### 1.4 Contraindications

Not known.

## 2 General safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

### 2.1 Warnings

- Enhance Multi Finishers and Polishers are designed for use with water. As with any rotary instrument, heat will build up with prolonged contact. Use at a speed between 5,000 and 10,000 rpm maximum with light intermittent pressure. Higher speed may result in heat buildup and may endanger the stability of the instruments causing danger to user and patient. Heat buildup may cause pulp inflammation.
- Care should be taken to have good control on any rotary instrument to protect patients from injury. Contact with soft tissues (skin, gingiva, mucosa) may result in injuries. If injury occurs, ensure appropriate medical care.
- Dental professionals should wear breathing protection (surgical mask). Dental professionals and patients should wear protective eyewear.

### 2.2 Precautions

1. This product is intended to be used as specifically outlined in the Instructions for Use. Any use of this product inconsistent with the Instructions for Use is at the discretion and sole responsibility of the practitioner.
2. Rotating instruments have to be disinfected, cleaned, dried and sterilized prior to the first application on the patient and immediately after every use.
3. Use in a well-ventilated area. Avoid inhaling airborne particulates.
4. Insert the instruments as deeply as possible without excessive use of force into the chuck and check to ensure that they are secure.
5. The hand piece must be in proper working condition.
6. The use of an adequate supply of water spray of a minimum of 50 ml/min on the working part is necessary over 1,500 rpm to ensure reliable cooling of the material being worked on and to prevent damage of the pulp.
7. Moistening of the surfaces to be prepared must not be impaired by an incorrect suction technique: Deflection of the coolant jet can cause heat related damage.
8. Protect adjacent restorations from accidental contact with rotating disc, cup or point.
9. Excessive speeds while using instruments with overly large heads endanger the stability of the instruments.
10. To maintain the longevity of these instruments, careful preparation with low contact pressure is important: the recommended contact pressure range is 0.3 - 2 N (30 - 200 p).
11. Avoid tilting, levering, or blocking the instruments up by using too much contact pressure (increased danger of fracture).
12. Immediately remove any instruments that are damaged, bent or no longer run concentrically.
13. The use of inappropriate disinfectants and/or prolonged disinfectant times can result in corrosion (increased danger of breakage) and discoloration of the instruments. Do not use alkaline or aldehyde containing disinfectants. Do not place in alcohol.

### 2.3 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

1. Dry storage is recommended.
2. Store at temperatures between 10 °C and 24 °C (50 °F and 75 °F).
3. Store the Enhance Multi Finishers and Polishers protected against contaminants and against chemicals, acids, heat, mechanical stress and extreme temperature variations.
4. Do not use after expiry date.

### 2.4 Adverse reactions

- Product misuse may cause failure resulting in respiratory, digestive, eye, skin, or mucosa injury. (See 2.1 Warnings and 2.2 Precautions)
- Product misuse may cause excessive heat. (See 2.1 Warnings)

## 3 Step-by-step instructions

1. Place restoration or composite veneer as per standard techniques. Complete gross reduction of excess and general outline form of the restoration using finishing burs.
2. Insert an Enhance Multi Finisher (dark grey) into a conventional speed contra-angled hand piece and continue finishing. Use with water. The aggressiveness of the Enhance Multi Finishers is controlled by the pressure applied to the surface of the composite. Use moderate to light, intermittent pressure in a buffing motion. Frequently raise instrument from surface contact.
3. Once final contours are achieved, proceed to polish. Insert an Enhance Multi Polisher (light grey) and continue polish for final high surface luster.

**Note:** Enhance Multi Finishing and Polishing Points are ideal for posterior occlusal and concave lingual surfaces. Enhance Multi Finishing and Polishing Cups are well suited to cervical and proximal line angle areas.

## 4 Hygiene

Sterilize prior to first use.

### 4.1 Cleaning

Remove surface debris by rinsing with water and brushing.

### 4.2 Disinfection

Disinfect with an aldehyde-free hospital-level, tuberculocidal disinfectant solution according to national/local regulations. The disinfectant manufacturer's instructions should be followed properly for optimum results. Remove residual disinfectant with water and dry.

### 4.3 Sterilization

Enhance Multi Finishers and Polishers are autoclavable.

Steam sterilization for 20 minutes at 134 °C (273 °F)/2.1 bar is recommended.

## 5 Lot number ( ) and expiration date ( )

1. Do not use after expiration date. ISO standard is used: “YYYY/MM”.
2. The following numbers should be quoted in all correspondence:
  - Reorder Number
  - Lot number
  - Expiration date

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© DENTSPLY DeTREY 2009-02-05
**DENTSPLY DeTrey GmbH**
**DENTSPLY Limited**
**DENTSPLY Italia S.r.l.**