Megalloy® EZ

DENSPLY
CAULK

Spherical Alloy Self-Activating Capsule

DIRECTIONS FOR USE – ENGLISH

For dental use only.
USA: Rx only.

Megalloy® EZ Spherical Alloy is a high strength, high copper, spherical alloy.

1. COMPOSITION

Alloy Powder	1 spill	2 spill	3 spill	Weight %
Silver	217 mg	326 mg	435 mg	56.7%
Tin	110 mg	164 mg	219 mg	28.6%
Copper	56 mg	85 mg	113 mg	14.7%
Mercury	295 mg	437 mg	593 mg	

The recommended alloy to mercury ratio by mass is approximately 1.3:1.

2. INDICATION FOR USE

Megalloy® EZ Amalgam is designed for use in stress bearing restorations (Class 1 and 2), when other restorative materials or restoration techniques are not indicated.

3. CONTRAINDICATION

The use of amalgam is contraindicated in patients with known allergies to amalgams or any of the components.

Safety alert symbol

This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

4. WARNINGS

1. It has been suggested by the Ad-hoc working group established by the European Commission that the following information, be kept in mind when considering the use of dental amalgams:

- After placement or removal of dental amalgam restorations increased mercury concentration in blood and urine has been observed. According to available scientific knowledge this increase has not been associated with any adverse health effects.
 - If placed in close contact with other metal restorations galvanic effects may occur. In most cases they will be of short lasting duration. If the effect persists the user should consider replacement of the dental amalgam filling with another material.
 - There are no proven adverse effects on the fetus associated with the placement or presence of dental amalgam fillings in the mother. It is sensible, however, where clinically feasible, to minimize health interventions during pregnancy and avoid any unnecessary chemical exposure of the fetus. This precaution should be observed with the use of all dental materials.
 - It has also been recommended by certain regulatory authorities that consideration be given to the use of amalgam fillings in children and that amalgams not be used in patients with severe renal disease.
2. Health Canada Warnings:
- Non-mercury filling materials should be considered for restoring the primary teeth of children where the mechanical properties of the material are suitable.
 - Whenever possible, amalgam fillings should not be placed in or removed from the teeth of pregnant women.
 - Amalgam should not be placed in patients with impaired kidney function.
 - In placing and removing amalgam fillings, dentists should use techniques and equipment to minimize the exposure of the patient and the dentist to mercury vapor and to prevent amalgam waste from being flushed into municipal sewage systems.
 - Dentists should advise individuals who may have allergic hypersensitivity to mercury to avoid the use of amalgam. In patients who have developed hypersensitivity to amalgam, existing amalgam restorations should be replaced with another material where this is recommended by a physician.
3. Alloy amalgam capsule products contain mercury, which can cause skin sensitization or other allergic reaction. Although the mercury in this product is bound within the amalgam following trituration, proper care should be taken to prevent exposure to mercury.

Skin Contact: Before using this product wear protective safety glasses, clothing and gloves. If contact with skin occurs immediately wipe off and flush with generous amounts of water, then wash well with soap and water after contact. If skin rash, irritation, sensitization or other allergic reaction occurs, discontinue use and seek medical attention immediately.

Eye Contact: May be irritating to eyes. Before using this product wear protective glasses as well as covering the patient's eyes to protect from excess. The use of an enclosed amalgamator is recommended. In case of accidental contact with eyes, rinse eyes immediately with plenty of water and seek medical attention.

Ingestion: Do not swallow or take internally. Whenever possible, the use of protective barriers such as rubber dam isolation and protective mask during placement and removal is recommended. If accidental swallowing occurs, drink lots of water. If nausea or illness develop, seek medical attention immediately. Contact regional Poison Control Center immediately.

Inhalation: Do not breathe vapor. Whenever possible, the use of protective barriers such as rubber dam isolation, high-volume evacuation and protective mask during placement and removal is recommended. Give oxygen or artificial respiration if necessary.

4. Do not open capsule prior to trituration. Do not remove or handle mixing pestle or plastic wafer containing mercury.
5. Do not attempt to clean and/or reuse capsules.

5. PRECAUTIONS

1. This product is intended to be used only as specifically outlined in the *Directions for Use*. Any use of this product inconsistent with the *Directions for Use* is at the discretion and sole responsibility of the practitioner.
2. Wear suitable protective eyewear, clothing, mask and gloves. Protective eyewear, and whenever possible, barrier techniques such as rubber dam is recommended for patients. (See *Warnings*)
3. Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse.
4. Mercury corrodes and embrittles particular metals and their alloys. Avoid unnecessary contact between mercury and metals.
5. Mercury presents a health hazard if incorrectly handled. Mercury is toxic by vapor inhalation and the effect is cumulative. Spillages of mercury should be removed immediately, including places which are difficult to access. Use a plastic syringe to draw it up. Smaller quantities can be covered by sulfur powder and removed. Avoid inhalation of the vapor.
6. Regulations for disposal must be observed. Adherence to the American Dental Association's current "Best Management Practices For Amalgam Waste (BMPs)" and "Dental Mercury Hygiene Recommendations" is strongly recommended. Among these practices:
 - Salvage and store non-contact amalgam scrap in well-sealed containers and recycle.
 - Salvage and store contact amalgam pieces from restorations, traps and filters in well-sealed containers and recycle.
 - Do not dispose of amalgam scrap, waste or extracted teeth in regulated infectious waste containers that will be incinerated, or in regular garbage.
 - Recycle used disposable amalgam capsules and plastic wafer (which contained mercury before mixing).
7. Waste material and all primary containers that hold mercury must be disposed of as hazardous waste.
8. If moisture is introduced into the amalgam before it has set, properties such as strength and corrosion resistance may be affected adversely. If the alloy contains zinc, such contamination may result in an excessive expansion (delayed expansion). Whenever it is possible, use a dry field.
9. Manufacturer's variation in amalgamators and differences in local electrical current may necessitate adjusting trituration time to obtain an optimal mix.
10. Insufficient data exist to support the use of amalgamators not listed in the chart below. Use of amalgamators other than those listed may result in less than optimally triturated amalgam. The operator must investigate non-listed equipment compatibility and/or consult amalgamator manufacturer for recommendations.
11. Do not attempt to place improperly triturated amalgam, or amalgam that has exceeded working time (dry, crumbly).
12. Drilling, polishing and grinding dental fillings should always be combined with water-cooling and suction under a vacuum.
13. The following precautions are required by the General Director AFSSAPS (Agence française de sécurité sanitaire des produits de santé – French agency for Sanitary Security of Health Products):
 - Store amalgam capsules in a cool and ventilated space.
 - Work in ventilated rooms with decontaminable non-textile surfaces.
 - Realize shaping and polishing of amalgam always under cooling and suction of the operation field.
 - Condense amalgam with classical methods (plugger) and do not use ultrasound condensers.
 - Do not place amalgam in direct proximity of other metallic restorations in order to avoid all risk of corrosion.
 - Avoid placing and removing amalgam during pregnancy and lactation.
 - If local reactions, in particular lichoid lesions in the proximity of amalgam, occur, it is justified to remove the filling.

6. STORAGE

Megalloy® EZ Amalgam in Self-Activating Capsules should be stored in a well ventilated place at temperature not to exceed 25°C/77°F. Do not allow exposure to moisture or excessive humidity. Do not use after expiration date.

7. ADVERSE REACTIONS

1. Product may irritate skin, eyes. **Skin contact:** irritation or possible allergic response. Reddish rashes may be seen on the skin. **Eye contact:** irritation and possible corneal damage. (See *Warnings and Precautions*)
2. Inhalation of vapors may cause varying degrees of damage to the affected tissue and also increased susceptibility to respiratory illness. (See *Warnings and Precautions*)
3. Product may cause neurotoxic, nephrotoxic or other serious health effects if inhaled or ingested. (See *Warnings*)

8. STEP-BY-STEP INSTRUCTIONS

- 1. Treatment of the preparation:** Clean freshly instrumented enamel and dentin with water spray and then air dry. The use of pulpal protection, bases and/or cavity liners is at the discretion of the operator, as dictated by the clinical presentation. If desired, Megalloy® EZ Amalgam may be placed over an adhesive liner or adhesively bonded by following the manufacturer's instructions for compatible adhesive systems (available separately).
- 2. Trituration:** Simply place Megalloy® EZ Amalgam Self-Activating Capsules in the amalgamator. These capsules are activated by the amalgamator – they do not require manual activation. Trituration speed and time is an important factor in preparing the amalgam, in order to achieve proper amalgam mix consistency. The chart that follows provides guidelines in determining amalgamator trituration time and speed setting for Megalloy® EZ Amalgam Self-Activating Capsules:

Approximate Trituration Time Range (in seconds)

Amalgamator	Speed	1 Spill	2 Spill	3 Spill
Caulk ProMix®	Rabbit (4200 cpm)	7-9	7-9	8-10
Caulk Vari-Mix® II M	M2 (3900 cpm)	7-9	7-9	8-10
Caulk Vari-Mix® III	M (3900 cpm)	7-10	7-10	9-12
Wig-L-Bug MSD	3800 cpm	8-11	8-11	10-12
Wig-L-Bug LPD	3800 cpm	9-11	9-11	9-11
*Kerr AutoMix	4000 cpm	10-13	10-13	9-12
*ADEC Model 6440	#7 (4200 cpm)	9-13	10-15	10-15
*Pelton & Crane Spirit Model SM	4000 cpm	12-15	12-15	13-16
*Zenith Stirspeed Model TP-103	Fixed 3950 cpm	8-11	10-12	10-12
*Zenith Stirspeed Model Z-1A	H 4100 cpm	7-11	7-12	10-13
*Vivadent Silamat Plus	Fast 4700 cpm	6-8	6-9	7-9
*Vivadent Silamat Plus 220V	Fast 4700 cpm	7-9	7-10	7-10
*SDI Ultramat 2	4400 cpm	8-12	8-12	7-10
*ESPE CapMix	Fixed 4400 cpm	8-10	8-10	9-12
*Torit Capmaster	4200 cpm	10-14	10-14	10-14

The following amalgamators are not recommended for use with Megalloy® EZ Amalgam in Self-Activating Capsules: *ESPE Rotamix and *Torit Single Speed Amalgamator Model TRA-18-1

*Not products of DENTSPLY International.

Suggestions for obtaining optimum mix

- 2.1 The triturated amalgam should be bright and have a plastic consistency. If the amalgam is excessively splashy and wet looking, INCREASE trituration time or amalgamator speed. Amalgam that appears excessively dry indicates trituration time or amalgamator speed should be DECREASED. Slow speed amalgamators are not recommended.
- 2.2 When performing adjustments, it is recommended to adjust trituration time in 2 second increments, until optimum mix is obtained. Greater adjustments may indicate amalgamator incompatibility or compromised functionality.
 Decrease trituration time: Increase trituration time:
 - To lengthen working time - To shorten working time
 - If mix is too hot - If mix is too wet
 - If mix is too dry
- 2.3 Properly triturated Megalloy® EZ Amalgam does not require mulling to produce a smooth plasticity; however you may mull for 1-2 seconds after removing pestle to collect amalgam into a single mass.

3. Condensation

- 3.1 No tool is needed to open the Self-Activating capsule. Just grasp both ends of the capsule, twist, and pull apart. Turn the opened capsule upside down to drop the amalgam mass into the work area. Megalloy® EZ amalgam is now ready for immediate placement and condensation. A gentle tap may be required to free amalgam from the capsule. Do not attempt to manually express mercury from the triturated mass prior to placement.
- 3.2 A plastic wafer and/or plastic mixing pestle may be seen. These have no further function and should be properly recycled.
- 3.3 Condense immediately after mix is completed. Use a clean amalgam carrier to transfer increments to the cavity. Pack each increment into angles and undercuts with small-faced pluggers and sufficient pressure to insure good adaptation. Build the restoration with additional portions, until the cavity is slightly overfilled. Remove any mercury-rich amalgam from the surface that may develop during condensation.
4. **Carving/Burnishing:** Carving can begin immediately after condensation. A sharp carver will give best results. The anatomy should be carved proceeding from the proximal margins towards the center of the restoration. Before the matrix band is removed, the marginal ridge should be supported by a condenser. After removal of the matrix band, gingival margins may be smoothed with a curved explorer. Burnishing setting amalgam is recommended to improve surface smoothness.
5. **Polishing:** The quality of the Megalloy® EZ Amalgam restoration will be enhanced by polishing. The amalgam should be allowed to set 24 to 48 hours before polishing. See selected polishing instrument manufacturer's directions for use.


Hygiene

- Cross-contamination
- Do not reuse single use products. Dispose in accordance with local regulations.

9. LOT NUMBER AND EXPIRATION DATE

1. Do not use after expiration date. ISO standard is used: "YYYY/MM"
2. The following numbers should be quoted in all correspondence.
 - Reorder number
 - Lot number on package
 - Expiration date on package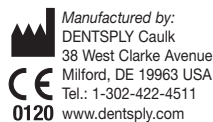

Distributed by:
DENTSPLY Canada
161 Vinyl Court
Woodbridge, Ontario
L4L 4A3 Canada

EC REP
DENTSPLY DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz, Germany
Tel.: 49-7531-583-0
www.dentsply.de

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