NUPRO[®]

PROPHYLAXIS PASTE

With Fluoride and Without Fluoride

CAUTION: United States Federal laws restrict this device to sale by, or on the order of a licensed dental professional.

These instructions cover all NUPRO Prophylaxis Paste Flavors/Grits.

NUPRO is a premium Prophylaxis Paste, available in fluoride-containing and fluoride-free formulations, which gently removes dental plaque and staining.

NUPRO Prophylaxis Paste is available in the following grits:

Fine (Gold)

Ideal for younger patients and prophyl procedures requiring minimal tooth abrasion.

Medium (Blue)

Suitable for most cleaning procedures where a high level of polish is desired.

Coarse (Red)

For patients with substantial stain build-up due to smoking or excessive use of stain producing beverages.

Plus (Green)

An extra heavy duty cleaner for stain build-up difficult to remove with conventional prophyl paste.

INDICATIONS FOR USE

To be used for cleaning and polishing procedures as part of a professionally administered prophylaxis treatment. Prior to pit and fissure sealing, NUPRO Prophylaxis Paste without fluoride should be used.

INGREDIENTS

Medicinal Ingredients: Sodium Fluoride (1.23% fluoride ion)

Non-Medicinal Ingredients: Pumice, Glycerin, Diatomite (fine grit only), Sodium Silicate, Methyl Salicylate, Monosodium Phosphate, Sodium Carboxymethylcellulose, Sodium Saccharin, FD&C Yellow #6 (fine grit), FD&C Blue #1 (medium grit) FD&C Red #40 (coarse grit), and FD&C Yellow #10 and Green #5 (plus grit), flavor, water.

CONTRAINDICATIONS

NUPRO Prophylaxis Paste with and without fluoride is contraindicated in patients with a known allergy to any of the components.

WARNINGS

- Excessive ingestion of fluoride can cause nausea. Therefore, pastes containing fluoride should be expectorated and not swallowed.
- NUPRO Prophylaxis Paste may cause eye irritation upon contact. If contact with eyes occur, immediately flush eyes with copious quantities of water for 15 minutes. Seek medical attention if irritation or discomfort persists.
- Avoid repeated skin contact with NUPRO Prophylaxis Paste, since repeated chronic exposure may cause dermatitis. If accidental contact with skin occurs, wash thoroughly with soap and water.
- Ingestion of NUPRO Prophylaxis Paste may cause salivation, nausea, vomiting. Ingestion of large quantities may cause abdominal pain, weakness, tremor, spasm or

ENGLISH

DENTSPLY
PROFESSIONAL

convulsions. Death can result from respiratory paralysis. If accidental ingestion of copious amounts occurs, call a physician at once. Immediately give copious quantities of milk, or water, if milk is not available.

- Unit dose cups contain enough Prophylaxis Paste for a single procedure. Each cup is intended to be used once for a single patient. Reuse of unfinished cup for another patient could result in cross-contamination.

PRECAUTIONS

- Prophylaxis pastes containing fluoride are not considered a replacement for other forms of fluoride therapy.
- NUPRO Prophylaxis Pastes are to be used only by individuals professionally trained to perform a dental prophylaxis.
- During use, wear protective glasses and gloves.

STEP-BY-STEP INSTRUCTIONS

When using bulk jars, sufficient paste for one procedure should be removed and placed in a clean dappen dish or other suitable container.

Unit dose cups contain enough Prophylaxis Paste for a single procedure. Each cup is intended to be used once for a single patient. Reuse of unfinished cup for another patient could result in cross-contamination.

Using a disposable prophyl angle such as NUPRO revolv[®] Disposable Prophyl Angle or a conventional metal prophyl angle, fill the cup by pressing the face of the cup into the paste while running the angle at low speed.

Throughout the procedure, sufficient paste should be maintained on the cup to avoid overheating of the tooth surface. Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste.

STORAGE

When using the bulk jars, close lid immediately after use to avoid moisture loss. Do not store above 25°C (77°F). Keep out of direct sunlight.

EXPIRATION DATE AND LOT NUMBER

The ReOrder Number, Lot Number, and Expiration Date should be referenced in all correspondence requiring identification of the product. Do not use after expiration date.

MATERIAL SAFETY DATA SHEET

is available on our website, www.professional.dentsply.com, by emailing your request to ProfessionalMSDS@DENTSPLY.com, or by contacting Customer Service at 1-800-989-8826.

Manufactured by:
DENTSPLY Professional
DENTSPLY International
1301 Smile Way
York, PA 17404 USA

Imported and Distributed by:
DENTSPLY Canada
Woodbridge, Ontario
L4L 4A3

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