DENTSPLY

Posterior Restorative

CAUTION

Cross-contamination

After intraoral use do not clean, disinfect or reuse Compules[®] tips.

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DENTSPLY

Posterior Restorative

QuiXfil™ is a resin-based restorative material specially designed for posterior use.

QuiXfil allows a simplified and fast application technique without compromising safety and efficacy.

QuiXfil utilises a new filler technology resulting in a unique high filler load.

QuiXfil exhibits wear resistance and physical properties suitable for occlusal stress-bearing restorations.

QuiXfil is available in pre-dosed in Compules® tips for direct intra-oral application.

QuiXfil is available in one universal shade.

QuiXfil is used following application of Xeno®III Single Step Self-Etching Dental Adhesive, Prime&Bond®NT, or other adhesives designed for use with QuiXfil.

COMPOSITION

Urethane dimethacrylate (UDMA)

Triethyleneglycol dimethacrylate (TEGDMA)

Di- and trimethacrylate resins

Carboxylic acid modified dimethacrylate resin

Butylated hydroxy toluene (BHT)

UV stabiliser

Camphorquinone

Ethyl-4-dimethylaminobenzoate

Silanated strontium aluminum sodium fluoride phosphate silicate glass

The filler : matrix ratio is 85.5 : 14.5 by weight and 66.4 : 33.6 by volume, respectively.

INDICATIONS

QuiXfil is indicated for:

- Class I and II cavities when safety and performance are to be combined with a fast and efficient application.
- Core build-up for metallic and PFM indirect restorations.

CONTRAINDICATIONS

- Use in patients with a known allergy to methacrylate resins or any other of the components.
- Direct or indirect pulp capping.
- When contamination with saliva, blood, etc. cannot be avoided.
- Core build-up for full ceramic crowns.

WARNINGS

1. QuiXfil contains methacrylates that may be irritating to the eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

2. QuiXfil may cause sensitisation by contact with skin or mucous membranes in susceptible persons. After accidental contact, wash immediately with plenty of soap and water or rinse with plenty of water. If sensitisation has occurred, discontinue use.

PRECAUTIONS

1. Use a gentle, even motion when exerting pressure on the Compules tip applicator gun. Use of excessive force or sudden movement could create a potential hazard.
2. Keep out of sunlight. To be stored at temperatures between 10 °C and 24 °C.
3. Humidity can adversely affect the properties of unsealed Compules tips. Therefore keep Compules tips sealed in their blister pack until use.

INTERACTIONS WITH DENTAL MATERIALS

Eugenol-containing dental materials should not be used in conjunction with this product because they may inhibit the curing of monomeric components.

STEP-BY-STEP INSTRUCTIONS

1. Preparation

Clean tooth to be prepared with a rubber cap and a prophy paste such as Nupro®.

Preparation may be kept to the minimum required for caries removal.

Finishing of cavity margins with a fine finishing bur will contribute to marginal quality.

After preparation, wash cavity thoroughly with air/water spray.

Remove rinsing water by blowing gently with an air syringe or blot-dry with a cotton pellet.

Do not desiccate the dentine structure.

2. Pulp Protection

For direct or indirect pulp-capping protect the dentine close to the pulp (< 1 mm) with a hard-setting calcium hydroxide liner (e.g. Dycal®), leaving the remaining cavity surface free for bonding with the adhesive.

3. Conditioning and application of adhesive

See instructions for use for Xeno III Single Step Self-Etching Dental Adhesive or Prime&Bond NT Nano-Technology Dental Adhesive.

4. Placement of QuiXfil

Insert Compules tip into the notched opening of the applicator gun barrel. Dispense QuiXfil directly into the cavity preparation. Due to the high depth of cure and low shrinkage for QuiXfil, placement in layers up to 4 mm thick is possible.

5. Curing

Cure each increment separately with a light curing unit:

- 20 seconds for units with an output of 500 to 800 mW/cm².
- 10 seconds for units with an output of 800 mW/cm² and more.

The tip of the light guide should be held as close as possible to the restoration during curing.

Important: Be sure to expose each area of the entire restoration to the curing light.

Additionally, the restoration should be cured through lingual or buccal enamel walls.

6. Finishing

Begin finishing immediately after curing. Gross excess material may be removed with fluted finishing burs or diamonds. Finishing is best achieved by using Enhance™ Finishing and Polishing Discs and interproximal finishing and polishing strips. In patients with an adequate oral hygiene, the final high lustre of the restoration develops with use.

MAINTENANCE OF APPLICATOR GUN

The applicator gun is sterilisable by autoclave or cold sterilisation solution following the manufacturers' instructions.

It is recommended that the applicator gun be disassembled for assured sterilisation. Partially close the applicator gun and place thumb under the

rear portion of the hinge. Push upward and lift hinge separating the applicator gun, exposing the plunger. Remove residual restorative with a soft paper tissue and a suitable solvent (70% alcohol).

To reassemble, insert plunger into applicator gun barrel, press components together and snap hinge mechanism in place.

STORAGE

QuiXfil Compules tips should be tightly closed immediately after use.

Keep out of sunlight.

To be stored at temperatures between 10 °C and 24 °C.

Inadequate storage conditions will shorten the shelf life and may lead to a malfunction of the product.

Humidity can adversely affect the properties of unsealed Compules tips.

Therefore keep Compules tips sealed in their blister pack until use. Under normal ambient conditions (18 - 24 °C, 50% ± 20% humidity) unsealed Compules tips stay usable for about 4 weeks.

BATCH NUMBER LOT AND EXPIRY DATE 

The batch number should be quoted in all correspondence, which requires identification of the product.

Do not use after expiry date.

If you have any questions, please contact:

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© DENTSPLY DeTrey 2004-12-07
DENTSPLY DeTREY GmbH