DENTSPLY

Silfix

Tray-Adhesive for A-Silicone Materials

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Silfix

Tray-Adhesive for A-Silicone Materials

Silfix is a polysiloxane-based adhesive which increases the adhesion between A-silicone impression materials and the impression tray surface.

Caution: To be applied by or under the control of a dentist.

COMPOSITION

Methylpolysiloxane

Toluene

Pigment (Solvent Green 3, Colour Index No. 61565)

INDICATIONS

Silfix is painted on the inner surface of the impression tray prior to application of the mixed impression material in order to increase adhesion to the tray of conventional, quadrofunctional and hydrophilic A-silicone impression materials (synonyms: Addition-curing silicones, siloxanes).

CONTRAINDICATIONS

Use in patients or by persons with a known allergy to any component of the product. Use with impression materials other than those listed in the Indications section.

WARNINGS

The product is:

- Highly flammable
- Harmful by inhalation
- Irritating to the eyes, mucosa, and skin.

Vapours may cause drowsiness and dizziness.

PRECAUTIONS

Use in a well ventilated place.

Do not empty into drains.

Keep away from sources of ignition. Do not smoke.

Take precautionary measures against static discharge.

Avoid contact with skin, mucosa, and eyes.

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

Avoid inhalation of vapours.

Keep container tightly closed.

INTERACTIONS WITH DENTAL MATERIALS

Adheres to unset A-silicone impression materials (addition-curing silicones, siloxanes).

STEP-BY-STEP INSTRUCTIONS

Application of Silfix

1. Select suitable impression tray.
2. Prior to mixing the A-silicone impression material, apply a thin coat of Silfix onto the inner surface of the tray. Do not extend over the border of the tray onto surfaces which will not be covered by the impression material.
3. Allow the Silfix layer to dry for at least 3 minutes to become tacky.
4. Load the tray with A-silicone impression material, e.g. Aquasil, in the usual way.

Cleaning of the tray

1. To remove any remaining adhesive after the cast has been removed from the tray, apply a solvent for removal of tray adhesives (e.g. Fix Solvent) onto the inner surface of the tray. Leave for some seconds to dissolve the remaining adhesive.
2. Wipe off residual solvent with paper towel.

STORAGE

Keep container tightly closed.

Keep away from sources of ignition – no smoking.

To be stored at temperatures between 10 °C and 24 °C.

Inadequate storage conditions will shorten the shelf life and may lead to a malfunction of the product.

BATCH NUMBER AND EXPIRY DATE

The batch number should be quoted in all correspondence which requires identification of the product.

Do not use after expiry date.

If you have any questions, please contact:

Manufacturer:
DENTSPLY DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
Phone +49 (0) 75 31 5 83-0

Distributor:
DENTSPLY Limited
Building 1
Aviator Park, Station Road
Addlestone, KT15 2PG
Phone: +44 (0)19 32 85 34 22

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Methylpolysiloxan

Toluol

Pigment (Solvent Green 3, Colour-Index-Nr. 61565)

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© DENTSPLY DeTrey 2003-10-06
DENTSPLY DeTREY GmbH