For better dentistry

DENTSPLY

ceram.x[®]one

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ceram.x[®]one

Universal Nano-Ceramic Restorative

CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	5
4 Hygiene	8
5 Lot number and expiration date	9

1 Product description

ceram.x[®] one restorative is a light-cured, radiopaque restorative material for restoration of anterior and posterior teeth. **ceram.x[®] one** restorative combines Nanotechnology with improved organically modified Ceramic particles, resulting in a Nano-Ceramic restorative with unique features: Natural esthetics by simple procedure, extraordinary low monomer release and superior handling characteristics.

ceram.x[®] one restorative provides two shading systems in one product to cover different clinical and esthetic demands:

ceram.x[®] one Universal restorative, the Single Translucency System, comprises seven shades of intermediate translucency comparable to conventional composites optimal for fast and easy restorations of anterior and posterior teeth.

ceram.x[®] one Dentin & Enamel restorative, the Double Translucency System, offers four dentin shades with translucencies of natural dentin and three enamel shades which mimic natural enamel. Their design has been optimized for highly esthetic restorations with a minimum number of shades.

The **ceram.x[®] one iShade label** eases shade selection, covering the whole VITA[®]1 classical shade range for both, the **ceram.x[®] one Universal** restorative and **ceram.x[®] one Dentin & Enamel** restorative system.

1.1 Delivery forms

- Syringes

¹ VITA[®] is a registered trademark of VITA Zahnfabrik H. Rauter GmbH & Co. KG.

1.2 Composition

- Methacrylate modified polysiloxane (organically modified ceramic)
- Dimethacrylate resins
- Fluorescent pigment
- UV stabilizer
- Stabilizer
- Camphorquinone
- Ethyl-4(dimethylamino)benzoate
- Barium-aluminium-borosilicate glass
- Silicon dioxide nano filler
- Iron oxide pigments, titanium oxide pigments and aluminium sulfo silicate pigments according to shade

1.3 Indications

- Direct restorations of all cavity classes in anterior and posterior teeth.

1.4 Contraindications

- Use with patients who have a history of severe allergic reaction to methacrylate resins or any of the other components.

1.5 Compatible adhesives

The material is to be used following application of a compatible (meth)acrylate-based dentin/enamel adhesive such as DENTSPLY adhesives designed for use with visible light-cured composite restoratives (e.g. the Spectrum® bond, prime&bond® one etch&rinse or prime&bond® one select).

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains methacrylates and polymerizable monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, red-dish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Tightly close syringes immediately after use.
- To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves. Discard syringe if contaminated.
- Interactions:
 - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.
 - If refrigerated, allow material to reach room temperature prior to use.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store in sealed packs at temperatures between 6 °C and 28 °C (43 °F and 82 °F).
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date.

Humidity can adversely affect the properties of the material. For unsealed syringes, the following is recommended:

- Store in a dry environment (< 80% relative humidity).
- Use within 3 months.

3 Step-by-step instructions

3.1 Shade selection

ceram.x[®] one restorative offers two different shade systems:

- ceram.x[®] one Universal restorative offers seven shades of intermediate translucency (a1-a4, c1, c2) and is especially suitable for fast, single-shade treatment procedures.
- ceram.x[®] one Dentin & Enamel restorative offers four dentin shades of low translucency (d1-d4) and three enamel shades of high translucency (e1-e3). With these shades you may copy every VITA[®] shade. ceram.x[®] one Dentin & Enamel restorative is especially suitable for esthetically demanding situations.

Decide whether you wish to work with ceram.x[®] one Universal restorative or ceram.x[®] one Dentin & Enamel restorative.

Concerning shade selection procedure there are two possibilities. You can use:

- The VITAPAN^{®2} classical shade guide together with the shade identification label (iShade label). With the enclosed iShade label you may select the suitable ceram.x[®] one Universal or Dentin & Enamel shade(s) corresponding to the VITA[®] shade range.
- The ceram.x[®] one shade tabs which are made from original ceram.x[®] one restorative material.

3.1.1 Shade selection with VITAPAN[®] classical shade guide and iShade label

- Fix the delivered iShade label at the rear side of the VITAPAN[®] classical shade guide.
- Select the VITAPAN[®] classical shade to be matched.
- Select the corresponding ceram.x[®] one shade according to the recipe on the iShade label.

ceram.x[®] one Universal restorative

Each VITA[®] shade is reproduced by one of the seven ceram.x[®] one Universal shades.

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ceram.x[®] one Dentin & Enamel restorative

Each VITA[®] shade is reproduced by a combination of one dentin shade and one enamel shade.

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3.1.2 Shade selection with delivered ceram.x® one shade tabs

To select the tooth color you may use the delivered shade guides: one of each is available for ceram.x® one Universal restorative and ceram.x® one Dentin & Enamel restorative.

Avoid contamination of the shade guide.

To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder. Always make sure individual tabs are completely reprocessed before placing back into shade guide.

ceram.x® one Universal restorative

1. Select the ceram.x® one shade most closely matching the color of the tooth. Consider the central region of the tooth.

ceram.x® one Dentin & Enamel restorative

1. Select the dentin shade most closely matching the color of the moist dentin visible in the cavity or consider the cervical region of the tooth.
2. Select the enamel shade most closely matching the color of the tooth. Consider the incisal, occlusal or proximal region of the tooth.

3.1.3 Shade selection hints

1. Clean the tooth surface with prophylaxis paste to remove plaque or surface stain.
2. Select the shade while teeth are hydrated.
3. When working with ceram.x® one Dentin & Enamel restorative you should select a slightly darker rather than a lighter dentin shade. The succeeding enamel layer tends to lighten the optical result.

3.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro® prophylaxis paste).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

3.2.1 Matrix placement and wedging (Class II restorations)

1. Place a matrix (e.g. AutoMatrix® or Palodent® Plus matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine® ring placement is recommended.

3.3 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/ or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

3.4 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 2 mm depth/thickness.

Avoid contamination of the syringe.

To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

1. Remove the cap.
2. Turn the handle of the syringe slowly in a clockwise direction and dispense the necessary amount of the material onto a mixing pad.
3. Point the front tip of the syringe upwards and turn the handle anti-clockwise to prevent oozing of the material.
4. Immediately close the syringe with the cap.
5. Protect the restorative material on the mixing pad against light.

3.4.1 Applying ceram.x[®] one Universal restorative

1. Fill the whole cavity incrementally. Each layer ≤ 2 mm.
2. Cure each layer according to the table below [3.5 Light-curing].

3.4.2 Applying ceram.x[®] one Dentin & Enamel restorative

Rebuild the tooth with respect to its anatomical structure. Consider that depending on the thickness of the enamel and dentin layers, the optical result may vary from the selected VITA[®] shade.

1. Rebuild the dentin with the selected dentin shade (thickness of increment ≤ 2 mm).
2. Cure each increment of dentin according to the table below [3.5 Light-curing].
3. Rebuild the enamel layer with the selected enamel shade (thickness of increment ≤ 2 mm).
4. Cure each increment of enamel according to the table below [3.5 Light-curing].

3.5 Light-curing

1. Light-cure each area of the restoration surface with a suitable curing light designed to cure materials containing camphorquinone (CQ) initiator, i.e. spectral output containing 470 nm. Minimum light output must be at least 500 mW/cm², exposure time according to the shade as listed in the table below. Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.

Insufficient curing.

- Inadequate polymerization.
1. Check compatibility of curing light.
 2. Check curing cycle.
 3. Check curing output.

Curing time for 2 mm increments

Output	≥ 500 mW/cm ²	≥ 800 mW/cm ²
ceram.x [®] one Universal shades (a1-a4, c1, c2)	20 sec	20 sec
ceram.x [®] one Dentin shades (d1-d4)	40 sec	30 sec
ceram.x [®] one Enamel shades (e1-e3)	10 sec	10 sec

3.6 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance® finishing devices for additional finishing.
3. Prior to polishing, check that surface is smooth and free of defects.
4. For achieving a very high luster of the restoration, we recommend Enhance® and PoGo®, finishing and polishing instruments, Prisma® Gloss™ Regular and Extrafine polishing paste.

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

4 Hygiene

4.1 Individual shade guide tabs

NOTICE

Cross contamination.

Infection.

1. To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder.
2. Always make sure individual tabs are completely reprocessed before placing back into shade guide holder.

NOTICE

Wrong cleaning or disinfection method.

Damage to the device.

1. Do not clean or disinfect with aggressive agents (e.g. solutions based on chlorine, glutaraldehyde, phenolic solvents, iodophors, alcohol, orange oil, or acetone) as they may affect the color shade over time.
2. Always make sure individual tabs are completely reprocessed before placing back into shade guide holder.
3. Never autoclave the individual tabs.

4.1.1 Manual cleaning

1. Discard gloves according to local regulations.
2. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
3. Use a new clean pair of examination gloves.
4. Clean device with a soft brush, warm water, and an appropriate cleaning solution until free of visible contamination. Pay special attention to device seams and insertions.
5. Remove cleaning solution residue with a cloth soaked with water.
6. Dry device with a lint-free single-use cloth.

4.1.2 Manual disinfection

1. After cleaning [4.1.1 Manual cleaning] thoroughly wipe all device surfaces with a single-use cloth in combination with a water-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.
2. Remove disinfectant solution residue with a cloth soaked with water.
3. Dry device with a lint-free single-use cloth.

5 LOT number () and expiration date ()

1. Do not use after expiration date. ISO standard is used: “YYYY-MM”
2. The following numbers should be quoted in all correspondence:
 - Reorder Number
 - Lot number
 - Expiration date

© DENTSPLY DeTREY 2014-02-19

[These Instructions for Use are based on Master Version 01]

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² VITAPAN[®] is a registered trademark of VITA Zahnfabrik H. Rauter GmbH & Co. KG.
DENTSPLY DeTREY GmbH
DENTSPLY Limited