001 1562R38Z

CLEARFIL™ SE BOND 2 [Unit Dose]

Dental Universal Self-Etch Adhesive

ENGLISH INSTRUCTIONS FOR USE

I. INTRODUCTION

CLEARFIL SE BOND 2 [Unit Dose] is a two component, self-etch, light-cured bonding agent. The PRIMER allows simultaneous treatment of dentin, enamel, non-precious metals and metal oxides using one liquid. The product is provided for single-use.

II. INDICATIONS

CLEARFIL SE BOND 2 [Unit Dose] is indicated for the following uses:
[1] Direct restorations using light-cured composite resin restorations
[2] Savily seating as a pretreatment for indirect restorations
[3] Treatment of exposed root surfaces
[4] Treatment of hypersensitive teeth
[5] Intraoral repairs of fractured restorations
[6] Core build-ups using a light- or dual-cured core material
[7] Cementing inlays, onlays, crowns, bridges and veneers using a composite resin cement

III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylate monomers

IV. POSSIBLE SIDE EFFECTS

The oral mucosal membrane may turn whitish when contacted by the product due to the coagulation of protein. This is usually a temporary phenomenon that will disappear in a few days. Instruct patients to avoid irritating the affected area while brushing.

V. INCOMPATIBILITIES

[1] Do not use eugenol-containing materials for pulp protection or temporary sealing, since the eugenol can retard the curing process.
[2] Do not use hemostatics containing ferrous compounds since these materials may impair adhesion and may cause discoloration of the tooth margin or surrounding gingiva due to remaining ferric ions.
[3] When using hemostatics containing aluminum chloride, minimize the quantity; use caution to prevent contact with the adhering surface. Failure to do so might weaken the bond strength to the tooth structure.

VI. PRECAUTIONS

[1] The product may contain substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to methacrylate monomers or any other components.
[2] If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching or numbness, discontinue use of the product and seek medical attention.
[3] Avoid contact with the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect them in the event of splashing.
[4] If the product comes in contact with human body tissues, take the following actions:
-<If the product gets in the eye:
Immediately wash the eye with copious amounts of water and consult a physician.
-<If the product comes in contact with the skin or the oral mucosa:
Immediately wipe the area with a cotton pellet or a gauze pad moistened with alcohol and rinse with copious amounts of water.
[5] Use caution to prevent the patient from accidentally swallowing the product.
[6] Avoid looking directly at the dental curing unit when curing the product.
[7] Single use only. Avoid using the same applicator brush on different patients to prevent cross contamination. Discard the container and applicator brush after each use.
[8] Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers or any other components.

2. Handling and manipulation precautions

1. The product must not be used for any purposes other than specified in the instructions for use.
2. The use of this product is restricted to licensed dental professionals.
3. Use a pulp capping agent in a cavity close to the pulp or in the event of accidental pulp exposure.
4. To prevent poor performance or poor handling characteristics, observe the specified light-curing times and other handling requirements.
5. Clean the cavity sufficiently to prevent poor bonding. If the adhering surface is contaminated with saliva or blood, wash it thoroughly and dry before bonding.
6. PRIMER is applied to the adhering surface with a large syringe or a dispenser 2-3 seconds. Move the operating light away from the mouth or turn the light to prevent the applied PRIMER and BOND from being exposed to the operating light (the BOND may be cured using the light).
7. Dry sufficiently with mild air after treatment with PRIMER for 20 seconds.
8. Do not rinse after applying PRIMER.
9. If the treated surface is contaminated, wash it with water, dry, or clean with alcohol, and treat again with the product.
10. Apply PRIMER/BOND with the applicator brush immediately after snapping off the container cap.
11. Do not use the same applicator brush for PRIMER and BOND.
12. When snapping off the container cap, do not tilt the container to avoid spilling PRIMER/BOND.
13. Do not mix the product with other materials.

[Dental light-curing unit]

1. Low light intensity causes poor adhesion. Check the lamp for service life and the dental curing light guide tip for contamination. It is advisable to check the dental curing light intensity using an appropriate light meter at regular intervals.
2. The emitting tip of the dental curing unit should be held as near and vertical to the resin surface as possible. If a large air surface has to be light-cured, it is advisable to divide the area into several sections and light-cure each section separately.

3. Storage precautions

1. The product must be used by the expiration date indicated on the package.
2. The product must be refrigerated (2-8°C/36-46°F) when not in use, and should be brought to room temperature for more than 15 minutes before using. After taking out of the refrigerator, the product must be left standing until it comes to room temperature.
3. The product must be kept away from extreme heat or direct sunlight.
4. The product must be stored in a proper place where only dental practitioners can access.

VII. COMPONENTS

Please see the outside of the package for contents and quantity.
[1] PRIMER (Yellow container)
Principal ingredients:
• 10-Methacryloxydihydrogen phosphate (MDP)
• 2-Hydroxyethyl methacrylate (HEMA)
• Hydrophilic aliphatic dimethacrylate
• d-Camphorquinone
• Water
[2] BOND (Black container)
Principal ingredients:
• 10-Methacryloxydihydrogen phosphate (MDP)
• Bisphenol A diglycidylmethacrylate (Bis-GMA)
• 2-Hydroxyethyl methacrylate (HEMA)
• Hydrophobic aliphatic dimethacrylate
• d-Camphorquinone
• Initiators
• Accelerators
• Silanated colloidal silica
[3] Accessory
• Applicator brush (fine «silvers»)

VIII. CLINICAL PROCEDURES

A-1. Standard procedure I
Direct restorations using light-cured composite resin restorations
A-2. Savily seating as a pretreatment for indirect restorations
A-3. Treatment of exposed root surfaces
A-4. Treatment of hypersensitive teeth

[WARRANTY]
Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

A-1. Isolation and Moisture control
To avoid contamination of the treatment area from saliva or blood to produce optimal results. A rubber dam is recommended to keep the tooth clean and dry.

A-2. Cavity preparations
Remove any infected dentin and prepare the cavity in the usual manner. [NOTE]

A-3. Pulp protection
Any actual or near pulp exposure can be covered with a hard setting calcium hydroxide material. There is no need for a cement liner or base. Do not use eugenol materials for pulp protection.

A-4. Acid-etching of enamel
There is an adherent surface including uncut enamel, apply a phosphoric acid-etching gel (e.g. K-ETCHANT Syringe) to the uncut enamel, leave it in place for 10 seconds, then rinse and dry. If acid-etching to the cut enamel is preferred, it is optional to apply a phosphoric acid to the cut enamel in the same manner as described above.

A-5. Application of PRIMER
1. Snap-off the container cap and apply PRIMER to the entire cavity wall with the applicator brush, leave it in place for 20 seconds. Use caution not to allow saliva or exudate to contact the treated surfaces.
2. Dry the entire cavity wall sufficiently with mild air for more than 5 seconds until the PRIMER does not move. Use a vacuum aspirator to prevent the PRIMER from scattering. [CAUTION]
Do not rinse after applying PRIMER.

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